
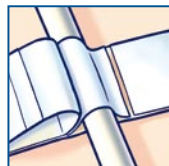
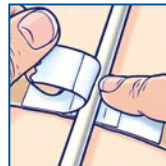


How to use Clinifix®

Clinifix® is simple and easy to use

1. Clean and dry the area (do not use liquid soap or other soaps with added moisturiser or lanolin). Ensure there is no moisturiser or cream present on the prepared skin surface.
2. Remove paper backing.
3. Place Clinifix® on the prepared skin surface. Smooth into place for 10-20 seconds to seal the hydrocolloid to the skin surface.
4. Release the Velcro®-type tab and place the tubing in a comfortable position.
5. Re-secure the Velcro®-type tab – for best result, do not stretch.

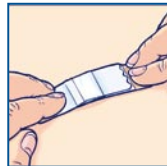

For extra security:

To secure the tube rigidly to restrict any movement, follow steps 1-4 and peel back the Clinifix® inner film (indicated by blue arrows). Place the tubing in a comfortable position on the adhesive and re-secure the Velcro®-type tab - for best result, do not stretch.

Clinifix® removal

1. Clinifix® can be left *in situ* for up to seven days.
2. Once Clinifix® has reached its full user wear time, it will peel off easily.
3. If Clinifix® is to be worn for a short period or on sensitive areas of skin, removal can be assisted by warm water or a medical adhesive remover.
4. Please note, prolonged contact with water (i.e. bath/shower) may reduce the wear time. Should the edge of the device come away from the skin, ensure the area is dry and simply smooth back into place.

